

# Lipo Slim

1 Year warranty on all equipment  
Extended warranties available  
Nationwide servicing  
Training provided



## Liposlim

The LipoSlim system uses the latest biological cavitation ultrasound technology combined with LED, effective for rejuvenation, to achieve unparalleled volume reduction in body tissue. The cavitation effect relies on the formation of bubbles within interstitial areas of the body. When these bubbles collapse the resulting shock wave leads to a temporary change in the permeability of the fat cell walls, which allows fatty material to drain out of the cell and be removed by the body's lymphatic system.

The power density and frequency of the LipoSlim system make it particularly effective for the destruction of consistent quantities of adipose mass most typically found in areas such as the abdomen, gluteus and thighs. After just one treatment it is possible to achieve a reduction in body circumference of up to 3cm. The LipoSlim allows quick pain-free treatments, excellent and stable results, and a rapid return on investment. It is also easily transportable between different locations because of its small size.

## Liposlim Procedure


A layer of gel is applied to the treatment area and an applicator is moved in smooth, circular sweeps, delivering the ultrasound energy. Two sizes of applicator are available so that treatments can be optimised to the different areas and shapes of the body. A warm, tingling sensation may be felt but treatment is very comfortable. Typical treatment times are in the range 15-25 minutes, depending on the area being treated.

### LipoSlim applications:

- Localised fat reduction
- Circumference reduction
- Improved appearance of cellulite
- Double chin reduction
- Improved skin appearance and texture
- Breast firming

### Features:


- Volume reduction without surgical intervention – no anaesthesia
- Visible results from just one session
- No patient down-time
- Easy to use
- Very low maintenance
- Very fast return on investment

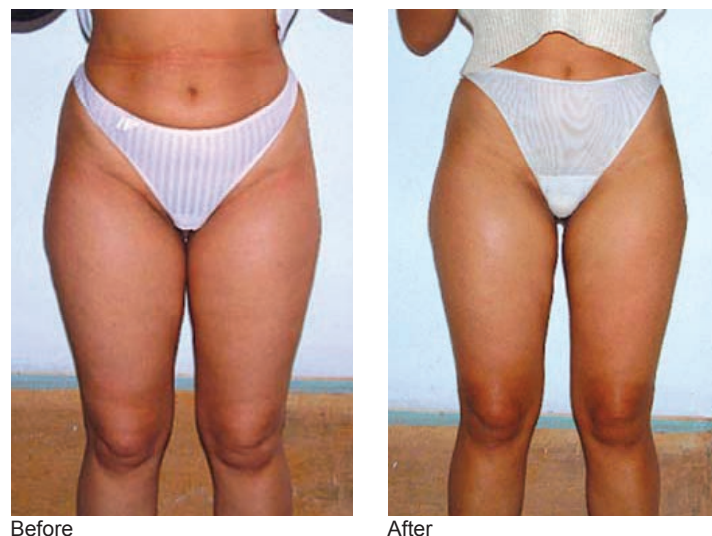

Before

After

## MONITORING WITH ELECTRICAL MICROSCOPE OF CAVITATION EFFECT IN A HUMAN TISSUE

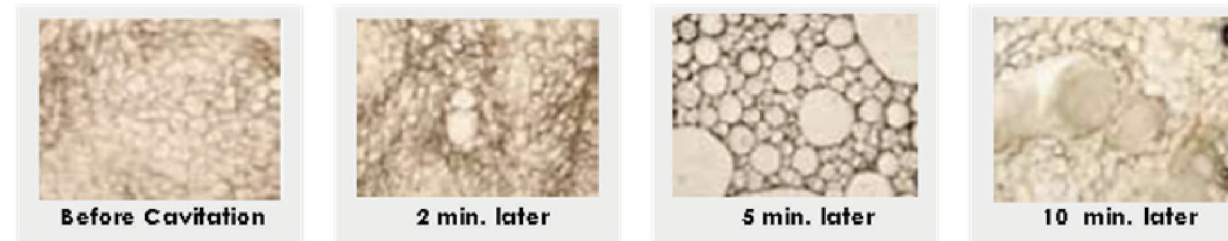


Before Cavitation

2 min. later

5 min. later

10 min. later


## STRONG CAVITATIONAL EFFECT BY LIPO SLIM (HANDPIECE 63Ø)

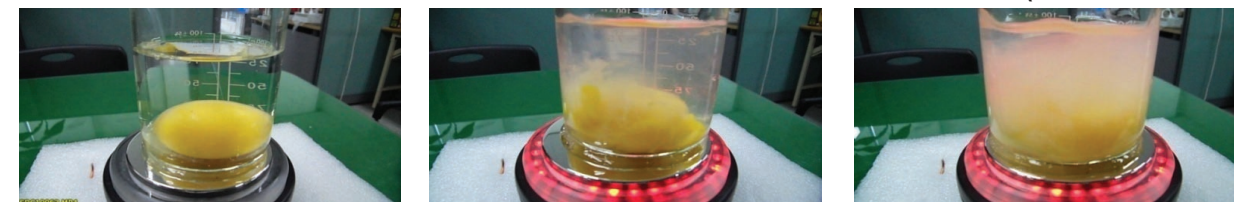

**Thermal Effect**  
Blood circulation  
Lymphatic drainage  
Metabolism

**Chemical Effect**  
Chemical reaction  
Collagen regeneration  
Skin p.H. balancing  
ATP activities

**Mechanical Effect**  
Massaging effect  
Fibroblast activities  
Elasticity of skin texture  
Softens connective tissue

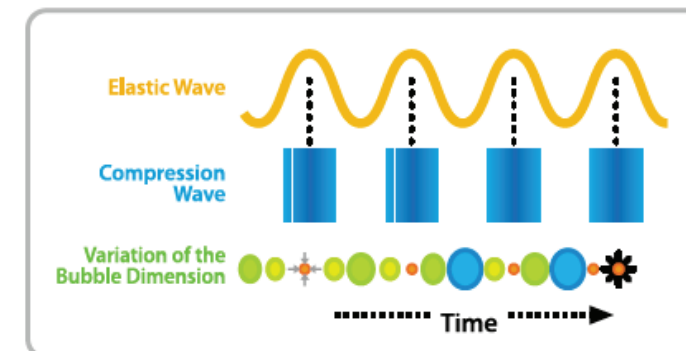

### Specifications:

Hand Pieces	:	63 Ø; 38 Ø
Ultrasonic Frequency	:	37 khz; 32 khz
Output Power	:	70 watt
Dimensions	:	243 x 290 x 358
Weight	:	5 kg

### Features:

- Colour Digital Touch Screen
- Intuitive Interface
- Preset Output Level
- Automated Preset Pulse



**PREFMED**  
PREFERRED MEDICAL TECHNOLOGIES

Distributed by Pref Med  
Tel: +27 021 791 2870 Fax: +27 021 791 2871  
www.prefmedtech.com

CE Department of Health License No.790/16543



**PREFMED**  
PREFERRED MEDICAL TECHNOLOGIES

Distributed by Pref Med  
Tel: +27 021 791 2870 Fax: +27 021 791 2871  
www.prefmedtech.com

CE Department of Health License No.790/16543